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Qu-resin

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Instructions for use

Qu-resin

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

Qu-resin is a quick-setting, self-curing denture repair resin based on diacrylate which hardens in three minutes.

2. Indication

Qu-resin is suitable for intraoral and extraoral (direct and indirect) use and available in pink and dentine colors.

3. Contraindication

Qu-resin is a resin based on methacrylates, inorganic fillers as well as initiators and catalysts. The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Hazard and safety information

Contains dibenzoyl peroxide. May cause allergic reactions:

S2: Keep out of the reach of children.

S13: Keep away from food, drink and animal-feed.

S15: Keep away from heat.

S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

S28: After contact with skin, wash immediately with plenty of water and soap.

S36: Wear suitable protective clothing.

If proper processing and use are ensured, undesired side effects caused by this product are highly unlikely. The product must not be used in case of hypersensitivity to methacrylate or one of the constituents listed above. Please refer to the safety datasheet for further safety information.

5. Storage and durability information

Storage:

The optimal storage temperature for Qu-resin is 15° C. The material must be stored in a way to protect it against sunlight and heat. After opening the container for the first time, Qu-resin should be used up within 3 months.

Please observe the storage information on the label.

Durability:

The product must not be used any longer once the shelf life has expired (see label).

6. Processing – use

Remove the cartridge cap (by turning it) and discard material until both pastes are dispensed evenly. To ensure perfect mixing of the material, use only the enclosed mixing cannulas. The mixing cannula in use is used to seal the cartridge until it is used again.

Use 110 µm aluminium oxide to sandblast the area to be repaired, degrease and wet with Qu-connector (observe instructions for use of Qu-connector). Then apply the respective material, pink or dentine-colored, and, if necessary use a spatula to spread the material. Ethyl alcohol can be used to clean the resin surface.

7. Symbols

Can be used until YYYY-MM

Observe instructions for use

Temperature limit

Lower limit _ °C

Upper limit _ °C

Protect from sunlight

Dry storage

8. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff. These user recommendations are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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