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MKZ Primer

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Instructions for use

MKZ Primer

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

MKZ Primer is a bonding agent for bonding metal and resin respectively ceramic and resin. MKZ primer is supplied in a plastic dropper bottle.

2. Indication range

- a) Conditioning of metal frameworks, for crowns, bridges, secondary frameworks, CoCr structures and abutments to be veneered with composite or resin (methacrylate-based) later on.
- b) Conditioning of oxide ceramic frameworks made of zirconium oxide; for crowns, bridges, secondary frameworks, CoCr structures and abutments to be veneered with composite or resin (methacrylate-based) later on.
- c) Conditioning of ceramic veneers for fixation or for repair with composite or resin (methacrylate-based).

3. Contraindication

Conditioning of resin dentures, acrylic teeth or natural teeth resp. substrate materials for ceramic veneering.

4. Safety recommendations and hazard warnings

MKZ Primer contains methacrylate. Sensitization by the product cannot be excluded for susceptible persons. In case of allergic reactions or known allergies to any of the constituents, such as resins based on (meth-) acrylates, the product should not be used. Avoid skin contact! In case of contact with eyes or extended contact with the mucous membrane, rinse immediately with copious amount of water and consult a doctor.

Relevant R-codes:

- R 11 Readily flammable
- R 36 Irritates the eyes
- R 43 May cause sensitization by skin contact
- R 66 Repeated exposure may cause skin dryness or cracking
- R 67 Vapors may cause drowsiness and dizziness

5. Storage and durability

The product must be protected against heat and stored at 4 - 25° C. MKZ Primer must not be used after the expiration date any longer.

6. Processing / Use

6.1 Conditioning of metal frameworks

Sandblast the metal framework at a pressure of 3 to 4 bars using 110 µ corundum (aluminium oxide) and clean it using air jet or a brush.

Note: Contamination with oil, separating agent or moisture (e.g. by touching the object with the fingers) must be avoided.

Add drops of MKZ Primer onto a mixing plate or in a mixing dish, apply to the framework using a disposable brush and let it dry for approx. 30 sec.

6.2 Conditioning of zirconium oxide frameworks / ceramic frameworks

Sandblast the framework, crown or bridge at max. 2 bars (activation) using 110 µ corundum (aluminium oxide).

Note: Contamination with oil, separating agent or moisture (e.g. by touching the object with the fingers) must be avoided.

Add drops of MKZ Primer onto a mixing plate or in a mixing dish, apply to the framework using a disposable brush and let it dry for approx. 30 sec.

6.3 Applying veneering material

First apply a thin layer of resin or composite, for example opaque material, immediately after conditioning and polymerize in accordance with the respective instructions for use. Adequate wetting of the surface must be ensured.

6.4 Intraoral repair of damaged crowns or bridge veneers made of metal or oxide ceramic

Place rubber dam. Sandblast framework and veneer surface carefully or roughen with diamond-coated grinder. Use a brush to apply MKZ Primer to the framework and adjacent veneer, wait 15 sec and coat the framework with opaque / adhesive liner and polymerize in accordance with the manufacturer's instructions. Use veneering material to complete the veneer and polymerize.

Note: it is mandatory to use rubber dam and avoid direct contact with the mucous membrane. In case of contact, rinse immediately with water.

7. Technical data

MKZ Primer exceeds the values of compressive and shear strength specified in DIN EN ISO 10477. See datasheet of the product.

8. Symbols

Xi irritant

F readily flammable

Observe instructions for use

Can be used until YYYY-MM

Lot number

Temperature limits (upper and lower limit: see durability and storage).

9. Additional information

The information contained in these instructions for use is always updated according to the latest knowledge and experience. Therefore we recommend to read the instructions for use again before using a new package. The product may only be used for the indication described under item 1.2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use. Any occurring claims for damages may only be made up to the value of our products.

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GmbH & Co. KG · Weissenhorner Str. 2 · 89250 Senden · Germany